



## K-D Test Pro Baseline iPad App

### Contents

Launching the K-D Test Pro Baseline Application .....	2
Log In .....	2
Administering the King-Devick Test .....	2
What is a Test Set? .....	2
Viewing Account Details.....	2
K-D Test Pro Baseline App Version .....	2
Log Out .....	2
Installing a K-D Test Pro Baseline App Update.....	2
Scoresheets .....	4

Your satisfaction is important to us. We're happy to assist you over the phone or via email.  
 Please contact us directly at: **+1-630-501-0281** or email us at [ProBaseline@kingdevicktest.com](mailto:ProBaseline@kingdevicktest.com).



## Launching the K-D Test Pro Baseline Application

1. Launch the K-D Test Pro Baseline App by tapping on the app icon:



## Log In

1. Log in with your K-D Test Pro Baseline account.
2. Tap 'Forgot Password' to retrieve your password. A working internet connection is required for this function.

## Administering the King-Devick Test

1. From the Home Screen, tap on 'Run Baseline Test' to initiate the K-D Test.
2. The instructional page appears. Read this in its entirety.
3. The test administrator will need a Scoresheet to keep track of errors committed by the subject.
4. Email Score Sheets to yourself by tapping "Scoresheets" or view Scoresheets on your smartphone at <https://www.kingdevicktestapp.com/phone>. A copy of the [Scoresheets](#) is also included in this manual.
5. You may select the Number of Test Cards and the Test Set to administer.

### What is a Test Set?

There are 3 test sets available. Test sets are the same King-Devick Test task with different number sets to minimize memorization. As the test administrator, be sure to have the corresponding Scoresheet set to follow along for errors.

6. Tap 'Start' to initiate K-D Test.
7. On the Results Screen, enter the total number of errors committed by the subject.
8. Follow on screen prompts to complete baseline testing. A minimum of two trials are required for K-D Test Baseline determination.
9. Upon completion of baseline testing, the K-D Test Baseline is displayed.
10. Tap 'Continue' to return to the Home Screen.

## Viewing Account Details

The Account Details menu allows you to view your account login, expiration date and [App Version](#). You can also [Log Out](#) of the app.

1. From the Home Screen, Tap 'Account Details'.
2. Tap 'Continue' to return to the Home Screen.

## Log Out

1. Under Account Details, tap 'Logout'.
2. Tap 'Logout' on the pop-up prompt.

## K-D Test Pro Baseline App Version

Ensure that you're running the latest version of the K-D Test Pro Baseline App for optimal performance. See [Installing a K-D Test Pro Baseline Update](#).

1. Under Account Details, view the App version in the bottom left (i.e. "V 1.0").
2. Tap 'Continue' to return to the Home Screen.



## Installing a K-D Test Pro Baseline App Update

Ensure that you're running the latest version of the K-D Test Pro Baseline App for optimal performance.

1. Launch the App Store app on your iPad.
2. In the Search Toolbar, search "K-D Test Pro Baseline".
3. If the App says 'Open', your K-D Test Pro Baseline App is up to date. If it says 'Update', tap 'Update' to download the latest version of the app.
4. To be sure that you're always running the latest version of the app on your iPad, tap 'Settings Icon' from your iPad home page. Then tap 'iTunes & App Store' and be sure that 'Updates' is turned ON.



## Scoresheets

### Test Set 1 Scoresheet

Answer Key Test Card I	Answer Key Test Card II	Answer Key Test Card III
2-5-8-0-7	3-7-5-9-0	5-4-1-8-0
3-7-9-4-6	2-5-7-4-6	4-6-3-5-9
5-3-1-6-4	1-4-7-6-3	7-5-4-2-7
7-9-7-3-5	7-9-3-9-0	3-2-6-9-4
1-5-4-9-2	4-5-2-1-7	1-4-5-1-3
6-5-5-7-3	5-3-7-4-8	9-3-4-8-5
3-1-8-6-4	7-4-6-5-2	5-1-6-3-1
5-3-7-5-2	9-0-2-3-6	4-3-5-2-7

Total Errors: \_\_\_\_\_

### Test Set 2 Scoresheet

Answer Key Test Card I	Answer Key Test Card II	Answer Key Test Card III
6-7-4-2-5	7-0-1-8-4	1-9-7-4-8
6-0-5-7-9	8-2-0-7-9	8-6-4-2-7
4-0-8-2-3	7-5-6-2-9	7-3-9-6-5
5-3-6-8-7	8-4-3-6-1	2-9-3-7-2
1-8-3-0-6	5-1-0-7-4	5-1-7-6-2
3-1-4-8-5	1-3-0-2-6	9-4-0-2-8
4-2-7-8-5	8-3-5-6-0	1-9-7-8-0
3-7-4-8-1	0-7-5-6-7	3-8-5-3-6

Total Errors: \_\_\_\_\_

### Test Set 3 Scoresheet

Answer Key Test Card I	Answer Key Test Card II	Answer Key Test Card III
7-4-8-1-6	6-9-3-4-8	2-4-8-3-1
9-8-3-5-2	5-0-7-2-3	7-0-6-5-2
8-6-4-0-1	0-6-1-8-5	1-3-9-4-0
3-7-2-5-9	1-3-9-2-3	8-2-7-0-5
5-6-0-3-2	2-7-5-6-4	5-8-6-9-7
8-1-5-2-0	8-6-1-0-7	4-0-2-5-3
4-5-3-1-7	9-4-2-8-5	9-6-3-1-8
1-8-2-3-9	1-2-6-0-3	7-9-5-4-6

Total Errors: \_\_\_\_\_



## Contact Details

### Corporate Headquarters

King-Devick technologies, inc.  
2 Mid America Plaza, Suite 110 Oakbrook Terrace,  
IL 60181,  
USA

Telephone: +1 844 606 7513  
Email: [support@kingdevicktest.com](mailto:support@kingdevicktest.com)

### European Authorized Representative

Waterfall Consultants trading as RPC Consultants,  
Ballyleigh,  
Waterfall,  
Co Cork,  
Ireland

Telephone: +353 21 239 4662  
Email: [kdt@rpcconsultants.com](mailto:kdt@rpcconsultants.com)

### Australian Authorized Representative

Neuropearce Australia  
PO Box 84  
Williamstown 3016

Telephone: +61 411 107 889  
Email: [ian@couttscommunications.com.au](mailto:ian@couttscommunications.com.au)

© 1983, 2010-2019 King-Devick Test. Patent Pending. All Rights Reserved.